


# Instructions

<u>Part #</u>	<u>Description</u>	<u>Packaging</u>
CT-120	Adult Breast Shield, Medium	each
CT-130	Adult Breast Shield, Large	each
CT-140	Pediatric Breast Shield, Neonate	10
CT-141	Pediatric Breast Shield, 1 yr old	8
CT-142	Pediatric Breast Shield, 5 yr old	6
CT-143	Pediatric Breast Shield, 10 yr old	4

## Instructions for use:

With the patient lying on the table in position for the scan, place the shield, foam side down, vinyl side up, over the breast area, with the notch towards the head as seen in the illustration below.

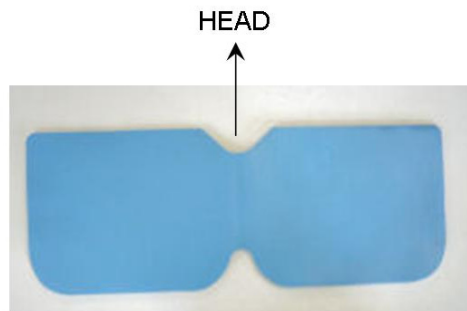

This can be done over the gown. Be sure to keep the gown smooth, do not wrinkle the gown.

NOTICE for Multichannel Helical Scanners with Dose Modulation. Before using the Breast Shield, it is important to run the scout topogram.

If the scanner detects increased density from the presence of the Breast Shield, It will adjust dosage to compensate as if the patient is larger. This may increase the patient exposure and defeat the shield purpose.

**AADCO Medical, Inc.**  
**PO Box 410**  
**Catamount Commercial Park, Route 66**  
**Randolph, VT 05060**  
Tel: 802-728-3400  
Fax: 802-728-3107  
Email: [info@aadcomed.com](mailto:info@aadcomed.com)  
Web: [www.aadcomed.com](http://www.aadcomed.com)