

LIMITED WARRANTY

This barrier and its accessories, excluding those listed below, are warranted by AADCO Medical, Inc., against defects in materials and workmanship for the life of the product. AADCO Medical will repair or, at its sole option, replace, at no charge, a barrier containing such defect provided it is returned, transportation prepaid, to AADCO Medical.

There are no warranties, express or implied, including without limitation any implied warranty of merchantability or fitness, which extend beyond the description on the face hereof. This express warranty excludes coverage of, and does not provide relief for, incidental or consequential damages of any kind or nature, including but not limited to loss of use, loss of sales or inconvenience. The exclusive remedy of the purchaser is limited to repair or replacement of the barrier at AADCO Medical's option.


This warranty does not apply if the product, as determined by AADCO Medical, is defective because of normal wear, accident, misuse, or as a result of service or modification by other than an authorized AADCO Medical repair facility. This warranty is void if the unit is subjected to temperatures above 55°C.

NON-WARRANTY SERVICE

If repairs or replacement not covered by this warranty are required, a repair estimate will be submitted for approval before proceeding with the repair or replacement.


REPAIR SERVICE: Return the product, prepaid, to:

AADCO Medical Inc.
 Catamount Commercial Park, Route 66
 Randolph, VT 05060

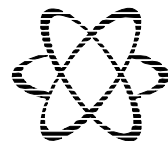

AADCO Medical Inc

**Mobile Barriers
 Instruction Manual**

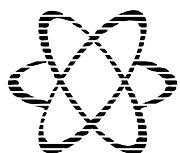

AADCO Medical, Inc.
PO Box 410
Catamount Commercial Park, Route 66
Randolph, VT 05060

Tel: (802)728-3400

Fax: (802)728-3107

Web: aadcomed.com

	Overall Size	Window Panel Lead Equiv.	Opaque Panel Lead Equiv.
Regular 0.5 mm S-602	75"H x 30"W	24"H x 30"W 0.5 mm	48"H x 30"W 0.8 mm
Regular Wide S-607	75"H x 36"W	24"H x 36"W 0.5 mm	48"H x 36"W 0.8 mm
Jumbo 0.5 mm S-606	75"H x 30"W	48"H x 30"W 0.5 mm	24"H x 30"W 0.8 mm
Deluxe 0.5 mm S-600	75"H x 48"W	30"H x 48"W 0.5 mm	42"H x 48"W 0.8 mm
Ultra 0.5 mm S-601	75"H x 72"W	36"H x 72"W 0.5 mm	36"H x 72"W 1.5 mm
Deluxe 1.0 mm S-605	75"H x 48"W	36"H x 48"W 1.0 mm	36"H x 48"W 1.5 mm
Ultra 1.0 mm S-604	75"H x 72"W	36"H x 72"W 1.0 mm	36"H x 72"W 1.5 mm

AADCO Manual S-602 Rev B © 2010

INTRODUCTION

Your Mobile Barrier is shipped to you partially assembled, in containers designed to minimize the possibility of shipping damage. Assembly is very simple. However, since the panels are heavy, they should be handled by two people until fully assembled.

CAUTION: Do not fully remove the protective paper from the viewing panel until after installation (See Step F for further instructions). This will help keep the leaded plastic from being scratched during assembly.


COMPONENTS SUPPLIED

- A. 1 ea. clear lead-plastic viewing panel
- B. 1 ea. opaque lower panel assembly with vertical side channels
- C. 4 ea. caster brackets, 2 with locking casters and 2 with non-locking casters
- D. 1 ea. 11MM Combination wrench
- E. 1 ea. 4MM Allen head wrench

ASSEMBLY PROCEDURE

A. To facilitate installing the lead-plastic viewing panel and caster brackets, lay the opaque lower panel flat on a table, with the mounting holes extending over the table's edge just enough to expose the bracket holes. (Fig. 1).

Caution: Do not let the panel overlap the table too far, to avoid the possibility of tipping the panel.

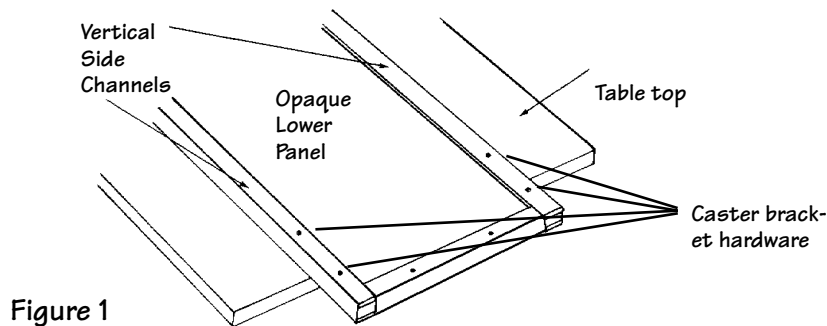

B. Remove the protective cardboard inserts between the vertical channels.



C. Inside the horizontal coupling channel (at the top edge of the opaque lower panel) is a strip of butyl tape with a protective cover. Peel back the protective cover and discard it.

D. Now, peel back the protective paper about two inches from the sides of the clear lead-plastic viewing panel. Carefully slide the panel into the two upright channels and press firmly into contact with the butyl tape.

E. Remove the Caster bracket hardware (pre-installed) from the four locations indicated in Figure 1 to ready the panel for installation of the caster brackets.

F. The pre-drilled holes in the caster brackets line up with the lower holes in the vertical side channels. Using the hardware removed in step E, (screws, washers and nuts), attach the brackets to the panel. (Fig. 3.)

NOTE: For maximum safety and convenience, the locking casters should be on the **same side of the panel.** (Fig. 2.)

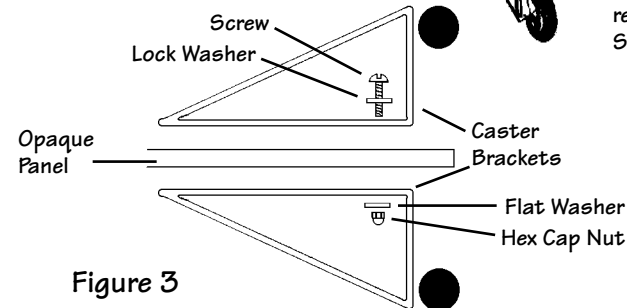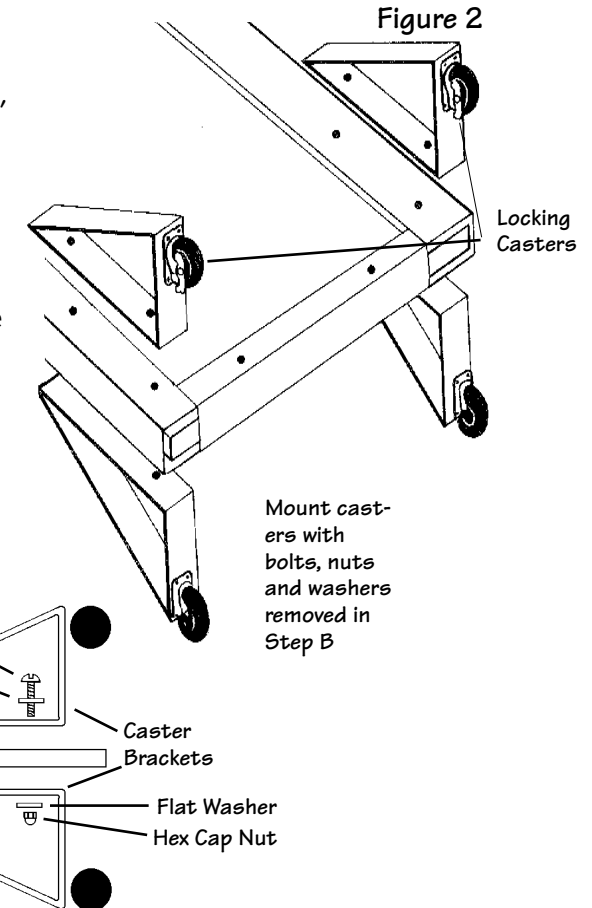

Carefully lower the assembly to the floor in a vertical position. Once upright, remove the rest of the protective paper from the clear panel. The barrier can now be moved about on its casters and is ready for use.