

# Assembly Instructions

## Mobile ErgoLite® Apron Racks Models R-400, R-401 & R-402.

### Contents:

- 1 - each hexagon wrench (Allen Key) for screws.
- 4 - long button head socket screws & washers.
- 2 - 'T' Sideframes with casters.
- 2 - Frame/hanger rods.

Note: Frame/hanger rod length varies by model, as follows:

- R-400 - 24"
- R-401 - 36"
- R-402 - 48"

### To Assemble:

#### Step 1


Remove the socket screws from the ends of one frame/hanger rod. Place one end of a frame/hanger rod in the larger hole at the base of the inverted 'T' frame and insert the socket screw and washer into the hole on the other side of the frame. Tighten the screw, leaving it slightly loose. Repeat this step on the other frame.

#### Step 2

Repeat Step 1, placing the remaining frame/hanger rod at the top of the inverted 'T' frames.

Side View

End View (half)

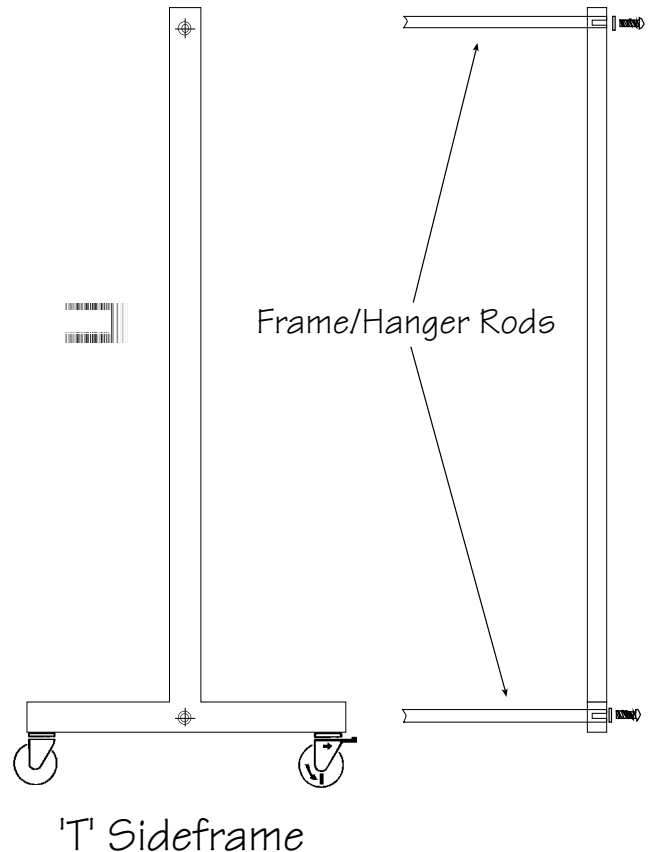

### ErgoLite® Apron Hanger

The special ErgoLite® apron Hanger combines a lightweight anodized aluminum hanger with a special bolster that fits under the shoulder of any ErgoLite® Apron, provides proper support for increased apron life.

R-E511

\$63.00

### ErgoLite® Replacement Supports

ErgoLite® Supports are made of high impact resistance Lexan®. Stress placed on supports in daily use can lead to the need for replacement. With reasonable handling, care and handling supports should perform well for a period of approximately one year. To maximize the effectiveness of the garment, supports should be replaced annually. Your ErgoLite® Apron has been designed to allow field replacement of the supports without returning the garment. You can do this simply by ordering a support replacement kit. Actual replacement should take less than five minutes and requires no tools.

PLP-E1	For 600 & 700 series garments	\$99.00
PLP-E2	For 200 series garment	\$99.00
PLP-EJ	"J" Series	\$153.00

AADCO Medical, Inc. Box 410, Randolph, VT. 05060 (802) 728-3400