

Assembly Instructions

Mobile Swivel Arm Apron Racks R-600 series

RayShield®/ErgoLite® Mobile Swivel Arm Apron Rack

This high quality rack combines a cantilevered design for superior stability with exceptional ease of access. Ideal for departments with limited wall space or where wall construction does not allow for wall mounted apron racks. Available with 12 regular hangers, 5 ErgoLite® apron hangers or a combination of 2 ErgoLite® apron hangers and 8 regular hangers. The rack size is 27" wide x 25-1/2" deep x 54" high. It's supplied in flat package, with minimal assembly required.

- Heavy Duty frame construction with enamel finish
- Choice of ErgoLite® or regular hangers
- Chromed steel pivoting apron hangers

R-600	12 Reg Hanger Rack
R-601	5 Ergo Hanger Rack
R-602	2 Ergo & 8 Reg. Hanger Rack



Contents:

- 1 - 5mm hexagon wrench (Allen Key) for M8 screws.
- 14 - M8 x 20mm long button head socket screws & washers.
- 2 - 'L' Legs with casters - parts (a) & (b).
- 1 - Lower frame tie - part (c).
- 2 - Swivel arm side stops - part (e) & (f)
- 1 - Top rack with 12 swivel arms for model R-600 - part (d) or ;
- 1 - Top rack with 5 swivel arms for model R-601 - part (d) or ;
- 1 - Top rack with 10 swivel arms for model R-602- part (d).

To Assemble:

Step 1

Attach the lower frame tie (c) to the 'L' Legs (a) & (b) with 4 M8 screws & washers.

Step 2

Attach the Top rack (d) to the 'L' Legs with 6 M8 screws & washers.

Step 3

Fix the swivel arm side stops (e) & (f) to the top rack with 4 M8 screws and washers.



Swivel arm side stops