

Installation Instructions


RayShield® Drift Resistant Trolley Assembly

Step 1: PREPARATION:

Make sure the inside of the track is CLEAN and DRY.



Step 2: Peel back approximately four inches of the protective backing on the Velcro strip. Place the Velcro strip around the installation roller with the Velcro facing inside the roller, the protective strip should be at the top of the installation roller and folded back.

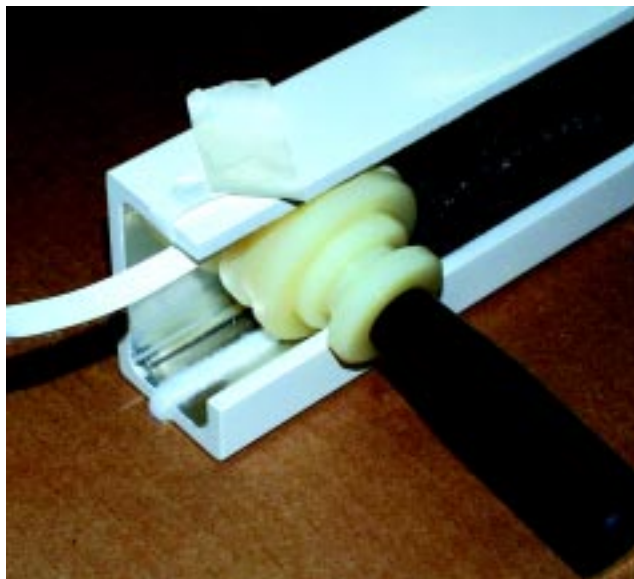

Step 3: Place installation roller inside track with approximately one inch of the Velcro folded onto the bottom of the track. Take the folded back protective strip and tape to the top of the track

Step 4: With the Velcro fasten to the bottom of the track, and the top protective strip taped to the top of the track. take the installation roller by the handle and roll out the Velcro strip into the bottom of the track.

Step 5: With your finger press down on the Velcro to make sure that the Velcro is fully adhered to the track.

Step 6: When you get to the other end of the track fold approximately one inch of Velcro around the bottom of the track just like the starting end.



Step 7: Repeat the same process on the other side.